Otto Bock®

QUALITY FOR LIFE

10A70=*, 10A80=*

Ⓓ	MovoHook 2Grip.....	3
ⒼⒷ	MovoHook 2Grip	5
Ⓕ	MovoHook 2Grip	7
Ⓘ	MovoHook 2Grip	9
Ⓔ	MovoHook 2Grip	11
⒫	MovoHook 2Grip	13
Ⓖ	MovoHook 2Grip	15
Ⓐ	MovoHook 2Grip	17
Ⓓ	MovoHook 2Grip	19
Ⓒ	MovoHook 2Grip	21
⒫	MovoHook 2Grip	23
Ⓒ	MovoHook 2Grip	25
Ⓖ	MovoHook 2Grip	27
Ⓔ	MovoHook 2Grip.....	29

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1 Application

The MovoHook 2Grip 10A70=* and 10A80=* are **solely** intended for exo-prosthetic fitting of amputations of the upper limb.

The hooks are designed for creating a cable-controlled arm prosthesis.

Cable-activated hooks feature:

- A rugged design that withstands the effects of forces and dirt
- The ability to grip even the smallest objects with precision
- Versatile gripping functions

Attention!

Use caution when handling the cable-activated hooks. The hooks have sharp contours. If used incorrectly, the hooks can cause injuries to you or others.

The ability of an upper limb amputee to drive a motor vehicle is determined on a case-by-case basis. Factors include the type of fitting (amputation level, unilateral or bilateral, residual limb conditions, or design of the prosthesis) and the amputee's individual abilities. For maximum safety and convenience, Otto Bock recommends that a specialist determine whether a vehicle needs specialised equipment, such as a steering fork. The patient must be able to operate the vehicle safely using the cable-activated hook.

2 Description and Function

10A70=* Cable-Activated Hook, light metal

10A80=* Cable-Activated Hook, stainless steel

All cable-activated hooks are available in the following versions:

*=LM12×1.5 left, with metric thread

*=RM12×1.5 right, with metric thread

*=L1/2"-20 left, with thread measured in inches

*=R1/2"-20 right, with thread measured in inches

A cable-control system is used to open the MovoHook 2Grip.

A pair of springs causes the hook to close automatically. The maximum gripping width is approximately 100 mm (4 in.). This enables the patient to grip large and cumbersome objects.

The pair of springs can be used to select two levels of grip force. If the spring lever is set proximally, the grip force is lower (Fig. 1, item A). This setting is intended for ordinary everyday use. It enables the patient to use less energy while working.

If the spring lever is set distally, the grip force is higher (Fig. 2, item A).

The recess in the area between both branches can be used for anchoring oblong objects, such as a file without a handle (Fig. 3, item A).

2.1 Maintenance and Repair

The MovoHook 2Grip does not require maintenance thanks to a new type of slide bearing. If necessary, worn out parts can be replaced by Otto Bock Service.

3 Liability

Otto Bock Healthcare GmbH, hereafter referred to as manufacturer, assumes liability only if the user complies with the processing, operating and maintenance instructions as well as the service intervals. The manufacturer explicitly states that this product may only be used in combination with components that were authorized by the manufacturer (see instructions for use and catalogs). The manufacturer does not assume liability for damage caused by component combinations which it did not authorize.

The product may only be opened and repaired by authorized Otto Bock technicians.

4 CE conformity

This product meets the requirements of the 93/42/EEG guidelines for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Otto Bock with sole responsibility according to appendix VII of the guidelines.

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Otto Bock HealthCare GmbH